

ENFIT™ ENTERAL SYRINGES

The ENFit™ enteral syringes are designed for delivering enteral tube feed, water, and medications via an enteral feeding tube, and/or to aspirate a feeding tube.

FEATURES

- **ENFit™ compliant:** improves patient safety by preventing misconnections with intravenous or Luer connections
- **Multiple syringe sizes:** 10ml, 20ml and 60ml syringes to support a variety of bolus feeding regimens and water flushing
- **Made from DEHP*, latex and PVC** free materials**
- **Single use and reusable options available**

Indications

Recommended for use in:

- Bolus enteral tube feeding.
- Administration of water flushes through feeding tubes.
- Administration of medications.
- Aspirating a feeding tube.

Directions for Use

- Administer feed and fluid in a controlled manner.
- Avoid using too much pressure or administering contents too quickly.
- When administering feed or fluid, if you experience a large amount of resistance, disconnect the syringe and consult a healthcare professional.
- Follow appropriate hygiene guidelines including hand washing and storage when using feed, tubes, feeding sets and syringes.
- Flush, clean and use/reuse your syringe as directed by your healthcare professional. The cleaning instructions for reusable syringes are to be used as a guide only.

Storage

- **Single Use Syringe[^]:** Store in clean, dry environment. Discard single use syringes as per manufacturer's instructions.
- **Reusable Syringe:** wash with warm soapy water, rinse and then dry syringe before storing in clean container with a lid between uses. See cleaning instructions for further information. Discard reusable syringes as per manufacturer's instructions.

Ordering Information

To order contact Nutricia Customer Care **1800 889 480**.

*DEHP - diethylhexyl phthalate

**PVC - polyvinyl chloride

[^]As classified by the Therapeutic Goods Administration (TGA) <https://www.tga.gov.au/>

For more information call the
Nutricia Clinical Care Line 1800 060 051



Images are not to scale.

PRODUCT	CODE	UNITS PER CARTON
Single Use ENFit Syringe 10ml	91421	30
Single Use ENFit Syringe 20ml	91422	30
Single Use ENFit Syringe 60ml	124823	30
Reusable ENFit Syringe 10ml	105969	30
Reusable ENFit Syringe 20ml	105968	30
Reusable ENFit Syringe 60ml	123584	30

Cleaning Instructions (reusable syringe only)

1. Do not allow contents to dry in the syringe before washing. Syringes can be cleaned by washing in warm, soapy water. The plunger should be removed and washed separately.
2. Do not scrub either component of the syringe as this will shorten the life of the syringe.
3. Rinse off each component with cold water thoroughly and allow to air dry. Do not dry the components with paper towel or similar as this will shorten the life of the device.

NUTRICIA
LIFE-TRANSFORMING NUTRITION

Nutricia Australia Pty Ltd.
Lvl 4, Building D, 12-24 Talavera Road,
Macquarie Park, NSW, 2113.
www.nutriciamedical.com.au

NC4654-Apr20 / NUT1141